ES - EN

P02  
Manual de instrucciones  
Instruction Manual  
WE 2021-P02-1

# INSTRUMENTO DE ALTA FRECUENCIA PORTÁTIL PORTABLE HIGH FREQUENCY INSTRUMENT

## ADVERTENCIA

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## ENGLISH

### WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

### EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

**1.** Conditions set out in the Low Voltage Directive: 2006/95/EC

**2.** Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

### RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

### WELCOME!

Thank you very much for purchase. Please read

the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

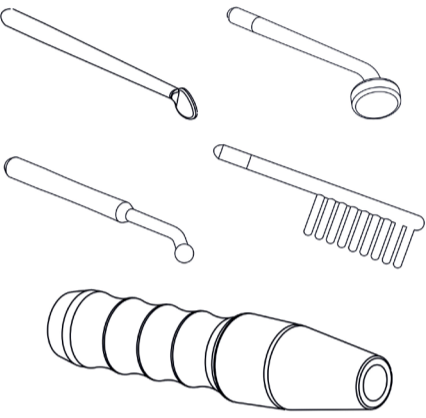
The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

### PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.


No.	PRODUCT	REF.	QTY
1	ELECTRODE 1	10009	1
2	ELECTRODE 3	10011	1
3	ELECTRODE 4	10012	1
4	SPECIAL ELECTRODE 3	10026	1
5	HIGH FREQUENCY HANDLE	-	1

### INSTRUCTIONS OF USE

**1.** Connect the power supply.

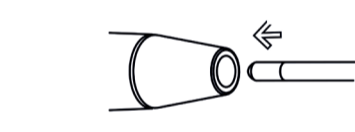**2.** Select the proper glass electrode pipe and insert the metal cap end into the output mouthpiece of the handle (try not to touch the glass electrode with your hands or any metal objects).

**3.** Touch the treatment area with the glass electrode. You can then slowly adjust the output using the regulator.

**4.** During the course of treatment, when changing electrodes, return the output to the minimum level before swapping the electrodes, then slowly adjust the energy level and return to the required intensity.

**5.** At the end of the treatment, slowly reduce the level to the minimum output before removing the electrode from the customer's face.

### INDIRECT USAGE

**1.** Select the straight glass electrode and insert the metal cap end into the output mouthpiece of the handle.

**2.** Give the handset to the customer, making sure they hold the glass electrode and not the handset.

**3.** Turn on the unit and make sure the regulator is set to the minimum.

### TECHNICAL SHEET

Weights and sizes	Packing size (cm)	32,5x21x6
	Gross weight (kg)	0,8
	Product size (cm)	23xØ4,5
Working characteristics	Working frequency	250 KHz
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz

**4.** Slowly adjust the output using the regulator.

**5.** Carry out the facial massage or treatment. Throughout the treatment process, make sure you do not touch or come into contact with the glass electrode being held by the customer.

**6.** At the end of the treatment, slowly reduce the level to the minimum output and then switch the machine off.

### CAUTIONS

• The glass electrode pole will sparkle when it is still less than one inch away from the skin with the instrument switched on. In this case, reduce the energy of the high frequency unit. If the customer still feels the sparkle, they may cover their face with a small, dry cotton towel but never with a nylon or synthetic one.

• Before removing the glass electrode from the skin, decrease the "ENERGY" to its minimum setting, then switch the power off before removing it from the skin.

• Customers should not be allowed to wear any metal objects or jewelry; all items must be removed prior to beginning any treatment.

• High-frequency treatments should not be carried out on pregnant women, or any persons using a pace-maker (or any electrical or rhythmic heart-regulating device).

• The products used with the high-frequency treatment must not contain any alcohol or spirit-based substances in order to prevent accidental combustion from arcing.

• Never touch the inner eye (eyeball) area with any high-frequency electrode.

• Only hold the handle of the high-frequency handset.

### MAINTENANCE

• Glass electrode pipes must be kept clean and dry, and should be sterilized before and after each use.

### ACCESSORIES

REFERENCE	DESCRIPTION
10009	ELECTRODE 1
10010	ELECTRODE 2
10011	ELECTRODE 3
10012	ELECTRODE 4
10024	SPECIAL ELECTRODE 1
10025	SPECIAL ELECTRODE 2
10026	SPECIAL ELECTRODE 3

Visit our website at [www.weelko.com](http://www.weelko.com) for more information

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