



PRODUCT INFORMATION

Ultrasound Transmission Gel, 5 l

ULTRASOUND gel has been designed to be used in ultrasound physiotherapy and diagnostic. The gel for ultrasound guarantees a perfect transmission of the ultrasound waves. It does not contain fatty substances, Easy to remove it doesn't stain, watersoluble, it does not irritate the skin, odourless. It does not contain any active principles that might damage the probes of the echograph equipment. It does not contain salt. Gel is colorless.

INSTRUCTIONS FOR USE

- * Gel is ready to use
- * Clean and degrease the patient's skin.
- * Apply the desired amount of Ultrasound gel
- * Connect and use the equipment and the probes by following the instructions supplied by the manufacturers.
- After the procedure remove all residues of the gel with paper, TNT (Non-woven) or gauze moistened with water.
- * Stop using the gel for hypersensitive patients, if skin reactions occur.

WARNINGS

- External use only.
- The product is not sterile.
- Do not apply gel on skin affected by abrasions, erythema or wounds.
- Close the bottle after use.
- * May cause irritations or reddenings with allergic persons.
- * Contact with eyes may cause irritation.
- * Analyst with sensitive skin should wear protective gloves during the use.
- * Do not use the gel after the "use by" date on the bottle.

INGREDIENTS: aqua, carbomer, sodium, hydroxide, EDTA, methylisothiazolinone, methylchlorisothiazolinone, benzyl alcohol.

PACKAGING

The product is supplied in the following packagings:

Carton with 4 canisters of 5000 g

DISPOSAL CONSIDERATION

Product, waste : Chemicals must be disposed of in compliance with the respective national regulations. Packaging must be disposed in compliance with the country-specific.

STORAGE

Temperature: +10°C ÷ +40°C - Relative humidity: 30% ÷ 75%. Keep container firmly closed, away from direct sunlight and sources of UV radiation.

EXPIRY

36 months from production date

Made in EU by Passo International order

Republikas laukums 3-24, Rīga, LV-1010, Latvia
info@beautyfor.lv

☎ +37167878628
💻 www.beautyfor.lv