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
I. Application

Portable rehabilitation tables enable the physiotherapist correct, safe and effective conduct of rehabilitation treatment, physiotherapy, physical therapy, massage, aiming at treating or alleviating the course of illnesses and the results of injuries /impairments. Used at the medical and doctor's offices, they allow patients to adopt the correct posture for conducting effective and safe treatment or examination. The construction of the rehabilitation tables enables the physiotherapist easy and complete access to the patient from each side, thanks to which all various treatments produce desired results and ensure proper conditions for the conduct of a treatment/examination.

II. Construction of the portable table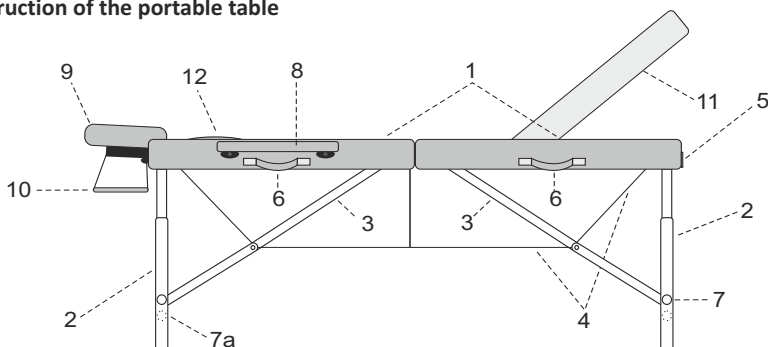

- | | |
|---|--|
| 1. Wooden table top covered with upholstery | 7. Locking pin |
| 2. External legs | 7a. Additional locking pin (regards Aura and Eden) |
| 3. External legs' spacers | 8. Armrests (regards Aura, Eden, Verona and Athena) |
| 4. System of cables | 9. Headrest |
| 5. Suitcase lock | 10. Arms front shelf (additional option) |
| 6. Transport handles | 11. Elevated backrest (regards stołu Eden, Vesta i Athena) |
| | 12. Face hole filler |

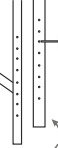
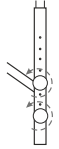
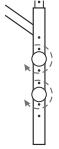
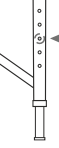
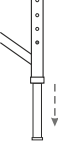
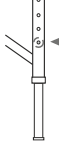
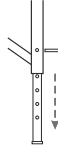
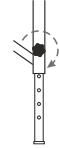
III. Operation

Sequence of unfolding the table:

1. Table in the folded condition constitutes a suitcase.
2. Open the lock (5).
3. Turn the table tops aside (1).
4. Draw out the equipment (8, 9, 10) from the table inside.
5. Unfold the legs outside (2) maximally extended.
6. Put the table on a hard and even foundation.
(Prior to loading the centre of the table top should be tilted up approx. 3 cm).
7. Check the tension of the system of cables (4).
8. Set the appropriate table height according to the table entitled "Height adjustment".
9. Install the headrest (9) and the remaining equipment (8, 10).
10. Lifting of the mobile part of the table takes place by means of:
 - a) unlocking elevated backrest,
 - b) lifting on the required height.*
11. Lowering of the mobile part of the table takes place by its maximum lifting and then lowering it down and locking.*


*regards Eden, Vesta and Athena

Tab. Height adjustment

1	2	3	Tables	Description of activities
			eSpirit	<ol style="list-style-type: none"> 1. Unscrew a knob. 2. Remove the mobile leg from the fixing screw, move the mobile leg to the chosen position. 3. Tighten the knob.
			Aura Eden Elza Olivia	<ol style="list-style-type: none"> 1. Unscrew a knobs. 2. Remove the mobile leg from the fixing screw, move the mobile leg to the chosen position. 3. Tighten the knobs.
			Verona Altus, Vesta Luna Light Luna Light Mini	<ol style="list-style-type: none"> 1. Drive in the locking pin. 2. Remove the mobile leg to the chosen position. 3. Drive out the locking pin.
			Torina Fairlane Thermo- Top Cinnamon	<ol style="list-style-type: none"> 1. Unscrew a knobs. 2. Remove the mobile leg from the fixing screw, move the mobile leg to the chosen position. 3. Drive out the locking pin.

V. Heated table top*

1. Turn on the table top heating by pressing the button ON/OFF on the remote control (1).
2. Set the requested temperature by pressing the buttons located below the HEAT '+' or '-' (2), in the range from 1 to 10. Heated table top warms up to a maximum temperature of 32° C.
3. Set the auto-off function by pressing the buttons located below the AUTO OFF '+' or '-'(3), in the range from 1 to 10. The digits represent the number of hours. By default, the heating will turn off after 1 hour. If we set 3, for example, the heating will automatically switch off after 3 hours, etc.

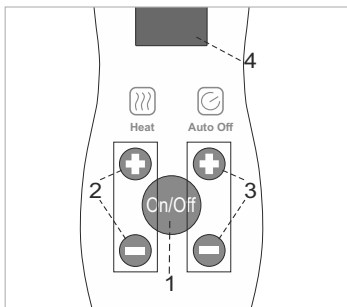NOTE:

When the heating in the tabletop switches off automatically, you can reset it by pressing the button ON / OF (1). If the letter 'F' displays on the remote control(4), disconnect the remote control from the table and the power supply, wait at least 20 seconds and reconnect it.

* regards table Thermo Top

VI. Laying the patient

The patient sits at the table side onto the upholstered top and adopts a comfortable horizontal position. The above-mentioned activities are safeguarded by the massage therapist, who determines the correct position on the table for the patients, and after the finished treatment helps them get down safely from the table.

VII. Folding the table down

1. Dismount the headrest (9) and the remaining accessories (8, 10) (if any).
2. Reset the table from the working condition to the folding position (lateral position on the rubber feet). Fold the external legs paying attention to the correct location of the cables and their fastening parts.
3. Put the equipment inside the table, close the suitcase by means of lock (5) and Place the suitcase in the carry case (if any).

VIII. Disinfection, maintenance and cleaning

1. **Lacquered elements.** For cleaning products that do not contain abrasives are recommended.
2. **Wood, plywood.** Surfaces made of wood or plywood should be cleaned by means of soft damp cloth wiping them until dry. No chemical agents should be used. Notice! Wood as a natural raw material may have differences in the tree ring layout, differences in the shades as well as natural inserts like: streaks, dots. Such properties do not constitute a basis for a complaint.
3. **Upholstery.**
 - Avoid contact with oils. To do this, use oil-resistant or waterproof cover sheets.
 - Upholstery should be used and stored away from direct sources of fire and heat.
 - Upholstery should not be exposed to long-lasting sunlight.
 - Do not use: pastes, waxes, sprays, strong detergents, agents containing solvents, cleaning agents for natural leather.
 - Dirt from oils, creams must be immediately removed with a mild detergent solution (warm water with potassium soap solution) and a soft cloth or sponge.
 - Daily soiling should be cleaned with a mild detergent, preferably potassium soap solution with a sponge or soft brush. At the end, wipe the cleaned place with a damp cloth and then wipe dry.
 - Stronger spot soiling should be cleaned using a 25% solution of ethyl alcohol. Wipe gently with soaked gauze swab. At the end, wipe the cleaned place with a damp cloth and then wipe dry.
 - Disinfect if necessary disinfectants dedicated to PVC, PU upholsteries. After disinfecting the tabletop or the mattress with any disinfectant thoroughly dry up the upholstery before inserting it into the carrying case. Otherwise upholstery could be permanently stained.

Attention: Before using other agent than mild detergent you need to check its influence on the place out of eyeshot. Cleaning should be carried out carefully.

IX. Exploitation remarks

1. The table is intended for use inside the dry and closed rooms and for the treatment of one person only at a time.
2. It is unacceptable to shift the table along with a patient on it. Rapid sitting on the table (jumping on), sitting, kneeling and basing on the headrest, shelves and armrests is forbidden .
3. While folding and closing the table attention should be paid to avoid snapping the cables.
4. The table should be used and stored far away from the direct sources of fire and heat.

X. Technical data

Table	Length	Width	Adjustable height	Dimensions after folding	Weight without accessories	Dynamic resistance*	Static resistance**
eSpirit	184 cm	64 cm 70 cm	58 - 79 cm 58 - 79 cm	92x64x20 cm 92x70x20 cm	13,0 kg 14,5 kg	270 kg	1000 kg
Aura	184 cm	64 cm	61 - 87 cm	92x68x18 cm	13,4 kg	250 kg	900 kg
Eden	184 cm	64 cm	61 - 87 cm	92x68x18 cm	14,5 kg	250 kg	900 kg
Elza	192 cm	71 cm	62 - 87 cm	96x71x18 cm	15,4 kg	250 kg	900 kg
Olivia	184 cm	71 cm	62 - 87 cm	92x71x24 cm	16,3 kg	270 kg	1000 kg
Verona	184 cm	71 cm	60 - 80 cm	92x71x18 cm	12,0 kg	250 kg	900 kg
Altus	184 cm	62 cm 71 cm	60,5 - 81,5 cm 60,5 - 81,5 cm	93x62x20 cm 93x71x20 cm	12,5 kg 13,5 kg	270 kg	1000 kg
Vesta	184 cm	71 cm	61,5 - 81,5 cm	92x71x20 cm	16,8 kg	250 kg	900 kg
Luna Light	184 cm	64 cm	61 - 81 cm	92x64x20 cm	10,3 kg	270 kg	1000 kg
Torina	184 cm	71 cm	61 - 81 cm	92x71x18 cm	15,5 kg	270 kg	1000 kg
Fairlane Therna-Top Cinnamon	183 cm	70 cm	58 - 84 cm	93x71x18 cm	15,4 kg	270 kg	1000 kg

*Dynamic resistance - measurement made with an evenly distributed dynamic load, simulating the tensions that occur during a typical classical massage.

**Static resistance - measurement made with an evenly distributed, fixed static load.

XI. Meaning of pictographs

Attention

Keep dry during storage

Manufacturer's name and address

Maximum load

CE mark - product compliance with the requirements of the Medical Devices Regulation (MDR) 2017/745/EU

See the instruction manual for use

Medical device

WARRANTY CONDITIONS

- For the purposes hereof:
 - „Seller" shall mean HABYS Sp. z o.o. located in Jasło, Poland, at 16 Produkcyjna Street.
 - „Purchaser" shall mean the final Purchaser of the equipment.
 - „Authorised Service" shall mean the service maintained by the Seller.
- The Seller shall ensure good quality and operation of the equipment for which the guarantee card has been issued for the period stated in the card.
- Guaranteed repair shall not cover activities provided for in the instruction manual which should be performed by the Purchaser himself.
- The guarantee shall not cover: damages to the equipment resulting from inappropriate usage or usage incompliant with the instruction manual, maintenance instructions, transportation instructions or additionally, any mechanical damages; colour differences between the parts exposed and not exposed to sunlight; impact of chemicals or abrasive agents, atmospheric pollution and accidental puncture, the presence of spots or rings on the oily or colouring substances (fats, creams, lotions or oils); permanent staining of some colouring agents, such as: ink, permanent markers or other non-colorfast dyes used in the production of clothing (for example: blue jeans).
- The Purchaser shall lose their guarantee rights if: the Seller discovers that during the guarantee period, the equipment was repaired outside the authorised service; any design modifications were introduced thereto without the Seller's consent; the equipment was still used following the disclosure of a constructional defect.

6. In cases provided for in point 5, irrespective of the loss of guarantee rights, HABYS Sp. z o.o. shall not be responsible for any damage to property or person which might be caused by operation of such equipment.
7. Defects or damages to the structure of the equipment resulting from the equipment itself, disclosed during the guarantee period, shall be removed free of charge within 14 days of submission of the equipment to the Seller or authorised service. Costs of transportation shall be borne by the Purchaser, if complaint is justified.
8. In order to enjoy guarantee rights, it shall be necessary to fulfill the following four conditions jointly:
 - a) Production of an invoice or other proof of equipment purchase by the Purchaser;
 - b) Reporting the fault within 14 days of disclosure thereof;
 - c) Sending back the damaged equipment in an original packaging.
9. The guarantee period shall be extended by the guarantee repair time, counted from the date of delivering the equipment to the Seller or authorised service to the day on which the equipment is made available to the Purchaser to recover.
10. Identification of the manner and necessary scope of the repair shall be the responsibility of the Seller or authorised service.
11. The Purchaser shall be entitled to, at their discretion, the right to claim equipment replacement or reimbursement of the price paid if:
 - a) The Seller has not enabled the Purchaser to recover the equipment repaired within 14 days of the Purchaser's delivering the equipment to the Seller or authorised service.
 - b) Within the guarantee period, four repairs have been made and the equipment still reveals defects making it impossible to operate it in compliance with its destination.
12. Equipment replacement or reimbursement of the price paid shall require the Seller's consent each time.
13. The Seller shall not undertake responsibility for damages to the equipment aroused during transportation by a forwarding company. It is the Purchaser's obligation to inspect the technical condition of the equipment upon the reception thereof.
14. If transportation was commissioned by the Seller and the goods have been damaged, the Purchaser is obligated to:
 - a) let the Seller know in 3 day time from the date of delivery about any damages of equipment, which happened during transportation in order to enable the Seller to file a complaint with the forwarding company.
 - b) provide the Seller protocol of complaint which is written between the Purchaser and transportation company.
15. Terms of guarantee contained herein shall be the sole and original terms of guarantee for Aveno Life products.
16. The warranty period for all Aveno Life products is 24 months, excluding: upholstered elements, actuators, electrical components - for which the warranty period is 12 months.

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