

Caution

1. To move this instrument, it is necessary to lift it up and put it down gently; in case of the failure, please do not knock hard on it so as to prevent malfunction caused by the impact.
2. To move this instrument, it is necessary to keep it horizontal so as to avoid vibration and impact.
3. Do not squeeze the instrument or place heavy stuff onto the instrument so as to prevent shell deformation, damage or malfunction.
4. It is necessary to select the corresponding working power supply in accordance with the marks on the instrument body.
5. It is not allowed to use this instrument in the bathroom.
6. To prevent the lightning accident, when it thunders, please show down the power switch and pull out the power plug.
7. It is not allowed to inject any fluid into the instrument body so as to prevent the instrument from being damaged due to the short circuit.
8. If the instrument will not be used for a long time, it is necessary to pull out the power plug from the socket.

Technical parameters

1. Operating voltage: AC 220V--AC 240V
2. Maximum power of the complete machine: 25W
3. Packing size: 26.5 *14*22.5 (L*H*W) cm

Version: 20140828

2 IN 1 Beauty Instrument User Manual



Dear customers,

Thank you for choosing our product! Please go through this manual before use in order to ensure the maximum performance of the instrument and the safety in use. Please keep properly for further requires.

Function

※ High Frequency

1. Make use of ultraviolet ray to do the irradiation on part or point of skin to sterilize the skin and accelerate wound healing.
2. Get rid of spots, pigmentation, pimples and acne


※ Galvanic

1. Thoroughly get rid of all dirt, impurities fat from deep pores of skin.
2. Nourish skin and improve skin conditions

Specification

1. Easy and fast operation by adopting latest microcomputer technology
2. Smart and reliable treatment by adopting 10 adjustable levels
3. Various methods available by adopting pulse wave

Operation Panel Description

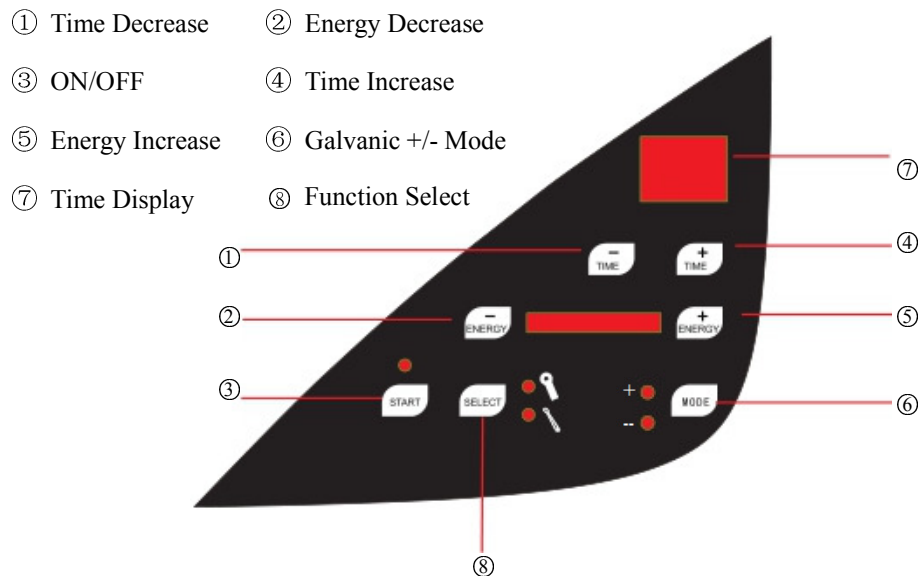

Operation

※ High Frequency

1. Test the function on some metal part before on client.
2. The energy should be not too high to avoid any injury or not too low to weaken the treatment result.
3. If for spot removal use, you need to move the electrode on the spot, then you will see the melanin decreasing. Tiny spot can be removed after one treatment, and bigger one will need 2-3 treatments.

※ Galvanic

1. Put the negative metal bar into the hand of client.
2. Dip lead-out medicine water / lead- in nourishing water into a small blade make-up cotton and stick to the roll gear of the Gal-bar, apply lead-out medicine water / lead-in nourishing water on the client's skin
3. Place the Gal-bar on the client's forehead;
4. Press button ⑥ to choose "+" for leading-out / "-" for leading-in
5. Increase/decrease the energy at 10 levels by slowly pressing button ⑤ / ②, treatment time at around 4~6 minutes.
6. After treatment, shut down the power and remove the galvanic head from client.

※ Operation

1. Plug in, ⑦ is displayed as 30 (minutes) and second point flashes
2. Press ⑧ to select the High Frequency / Galvanic function.
3. Press ① / ④ to adjust the work time
4. Press ⑤ / ② to adjust the energy.
5. Button ⑥ to choose "+" for leading-out / "-" for leading-in only when Galvanic on.

Notes

1. If beautician does the treatment with sterilized gloves, hands should first be cleaned and disinfected with alcohol
2. The electrodes should be sterilized, kept clean and dry, and fixed into the handle
3. Disinfect the treatment part of client with alcohol before treatment
4. Wipe part with anesthetics for 3 minutes and then start the treatment
5. During operation, not to stay too long in a treated area to avoid scars
6. Take off all metal parts (earrings, necklace etc.) before operation