



Instructions Manual

Electric Massage Bed
3897+.4.A26.HT

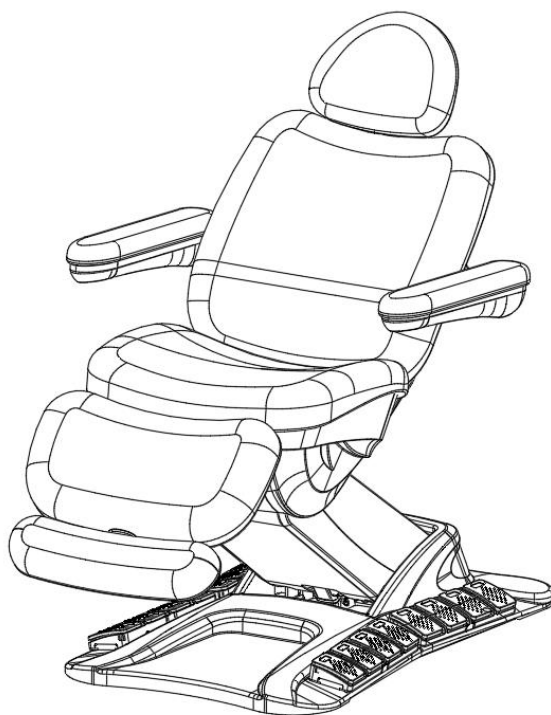

Safety instructions


1. Please read the following instructions carefully before connecting, operating or adjusting this electric beauty bed and keep this manual safe.
2. This electric beauty bed uses a three-wire power cable, which must be inserted into a three-hole power socket with earth wire to ensure proper ground connection so that it can prevent user from getting electric shock accidentally. Meanwhile it also ensures the normal operation of the electric beauty bed.
3. All plugs, buttons, and their connecting cables must not be immersed in water or other liquids during normal use.
4. It is forbidden to put your hands, feet and head into the gap ①②③④⑤⑥⑦⑧ (see the following figure) while adjusting the height of the entire bed by pressing the buttons on hand controller or foot controller.
5. It is prohibited to use immediately when plugs, buttons and their connecting cables are broken.
6. It should be noticed that the frequency is 6 min/h when using the electric push rod of this electric beauty bed.
7. Maintenance must be carried out by trained professionals.

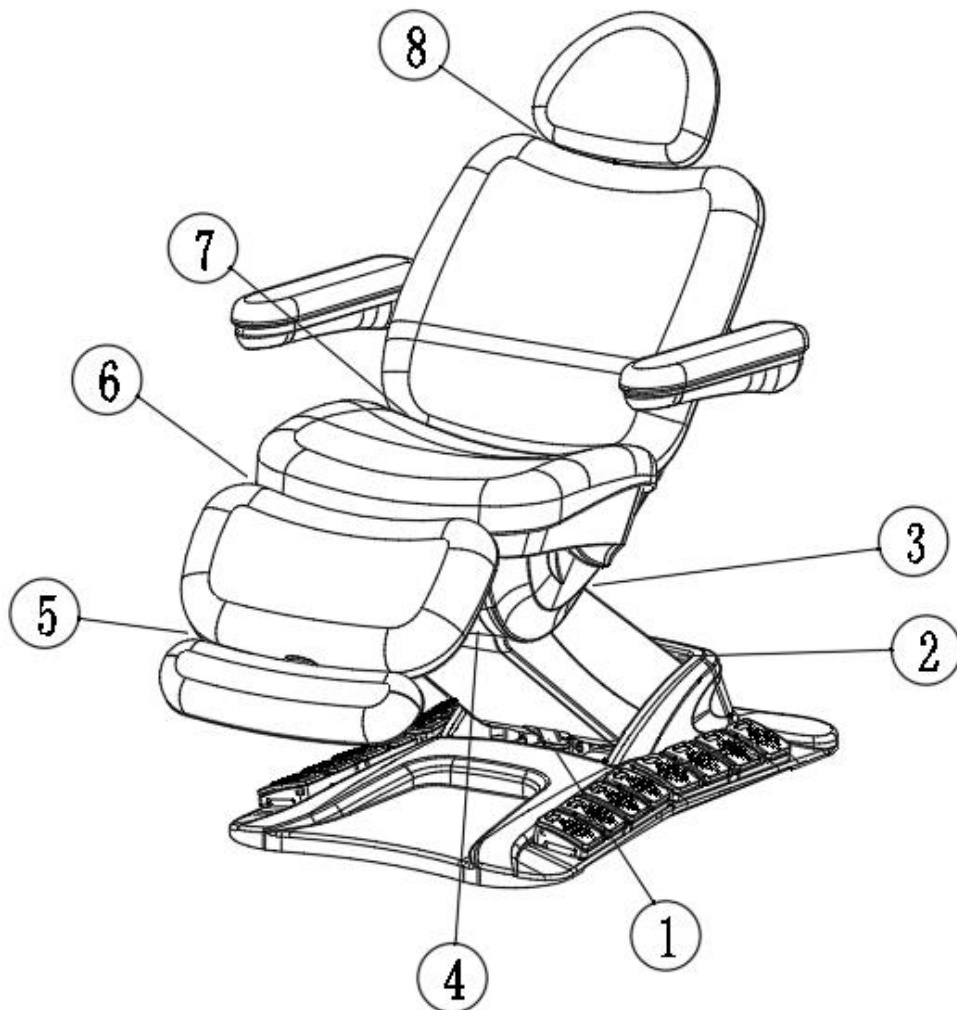


Warnings:

1. Individuals with osteoporosis or heart problems should not use this product;
2. Pregnant or menstruating women should not use this product;
3. Individuals with physical trauma or surface diseases should not use this product;
4. Children under 14 years old without any supervision and someone unconscious are prohibited from using this product;
5. Please check whether the power cord and plug are damaged or not before use. If there is any damage then it must be replaced by the agents or similar professionals;
6. Please confirm whether the voltage used accord with the safety voltage of this product or not before use as the incorrect voltage may cause damage to the product;
7. Wet hand is not allowed to plug or unplug the power plug to avoid electric shock;
8. Do not plug in before the product is installed;
9. The power cord should be put away to prevent pedestrians from tripping;
10. The appearance of the product should be cleaned by soft towel soaked with diluted neutral detergent;
11. Unplug the product when it's not used;
12. Do not disassemble the product without any authorization. Please contact the local agents for repair and maintenance.

• Technical data

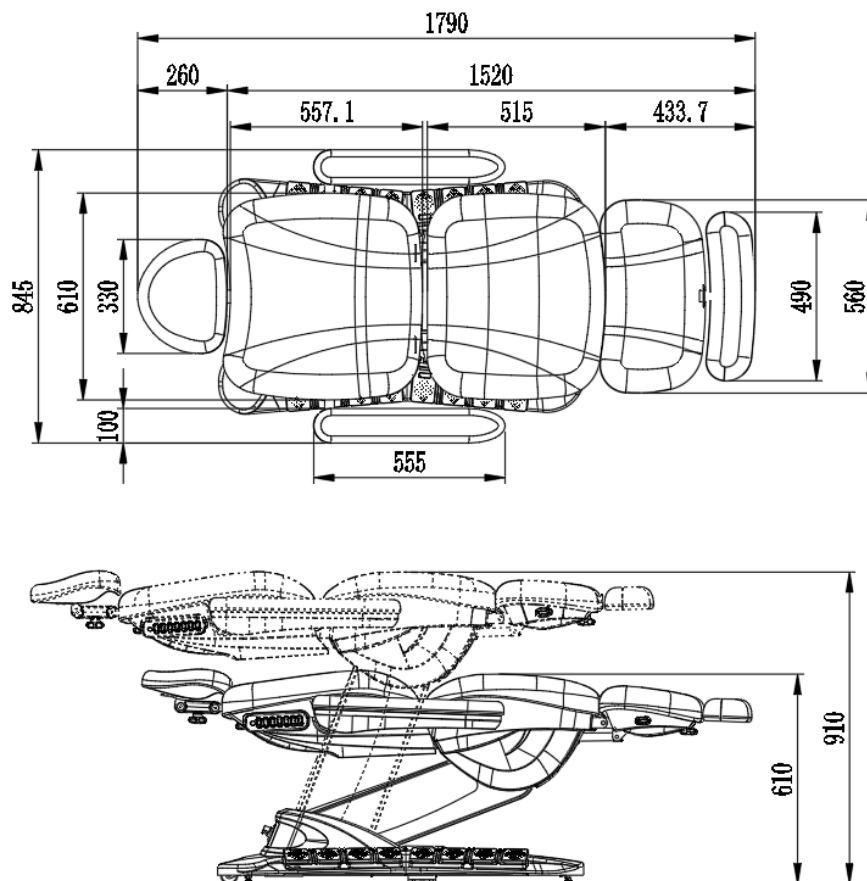


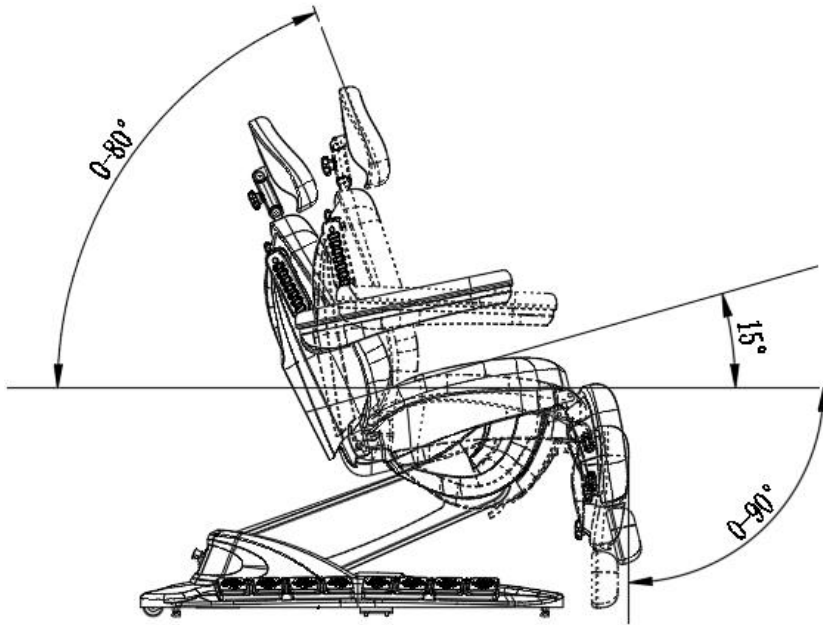

Dimension	1790x845x610mm
Lowest Height	610mm
Highest Height	910mm
Loading Capacity	200kg
Net Weight	86kg
Gross Weight:	106kg
Packaging size:	126*70*63cm
Input Voltage:	AC 220~240, 50/60Hz
Output Voltage:	DC 24V
Maximum Power:	160W
Relative Humidity:	45-85%
Environment temperature:	-5°C to +50°C
Insulation class:	Class II


● Assembly Instruction:

1. Power cord assembling: As below picture 1, plug in power cord, turn on power switch, all the buttons of foot and hand controllers will light up.
2. Armrest assembling: As below picture 2, line up and connect the left and right link of armrest and pin in the bed, then rotate 90° counter-clockwise.
3. Headrest assembling: As below picture 3, connect the headrest and bed, loose rotary knob of headrest for height adjustment.

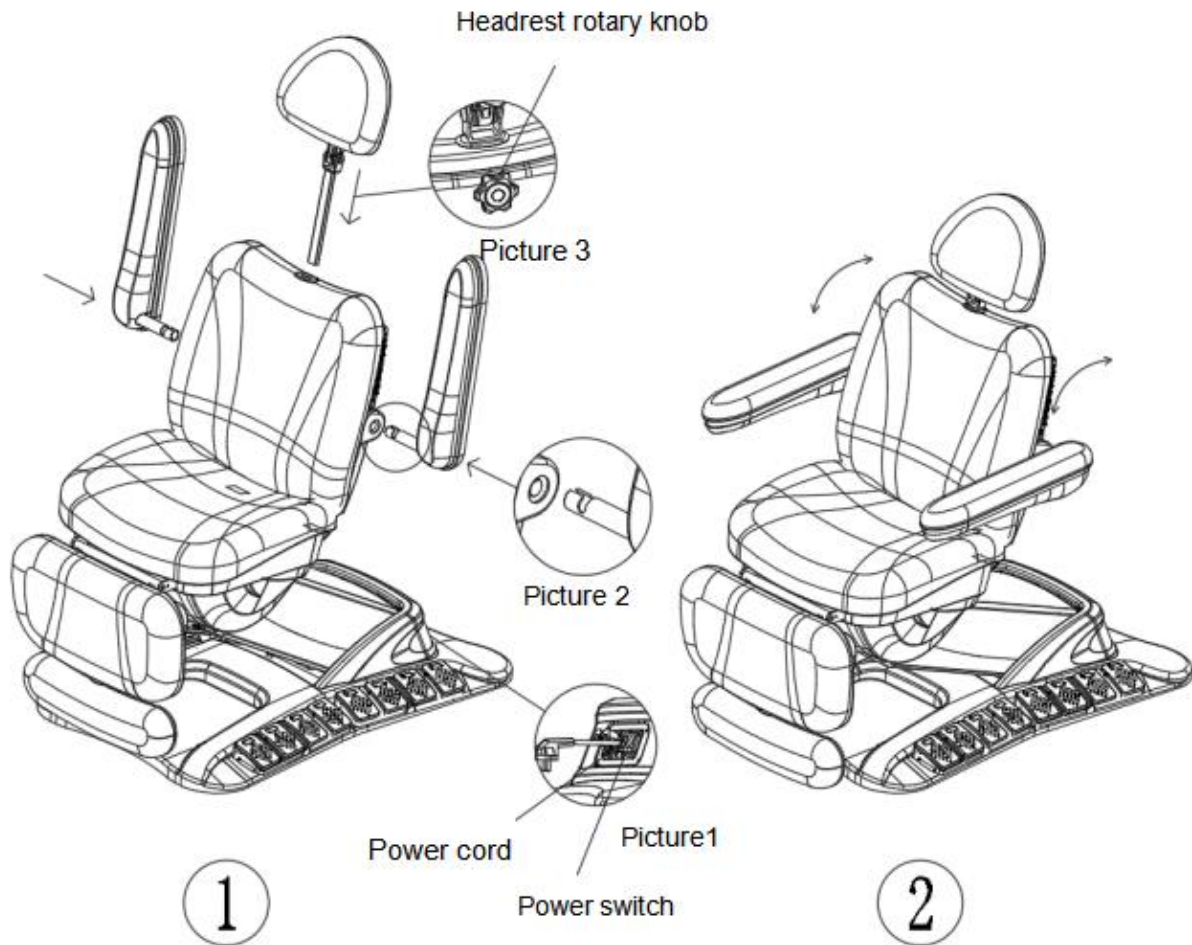

● Usage Instruction For Hand Controller:

- As below picture for hand controller:

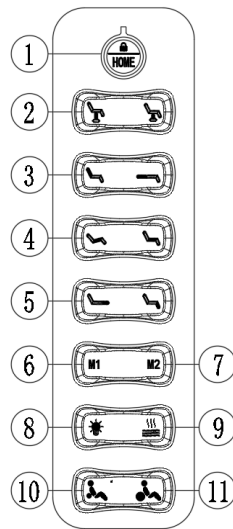

Button Instruction for hand controller:

Button ①: Manual lock button and reset button: Long press the lock button , all lights will go off and all the buttons are locked. Long press the lock button again for 3 seconds (light flash three times) to unlock the buttons; Press  to reset to the initial position.

Button ②: Control overall lifting of the bed.

Button ③: Control tilt angle of back cushion lifting.

Button ④: Control tilt angle of seat cushion.

Button ⑤: Control tilt angle of leg cushion.

Button ⑥⑦: Automatic mode button: The bed will be adjusted to preset height and angle automatically by pressing either of these two buttons. (Automatic mode setting: adjust lifting, tilt angle, back cushion to preferred position, then long press button M1 or button M2 for memory function setting)


Button ⑧: Coloured lighting button: The color of light will keep changing by pressing this button each time and when it comes to the 7th time pressing, the color of light will change circularly.

Button ⑨: Press this button, the cushions will be start heated to (40 °C±5 °C) after a beep sound; Press this button again, the cushions will be start heated to (50°C±5°C) after three beep sound; Press this button again to turn off cushion heating.

Button ⑩: Airbag button, long press this button to turn off airbag.


Button ⑪: Airbag button, long press this button to turn on airbag.

● Button Instruction for foot controller:

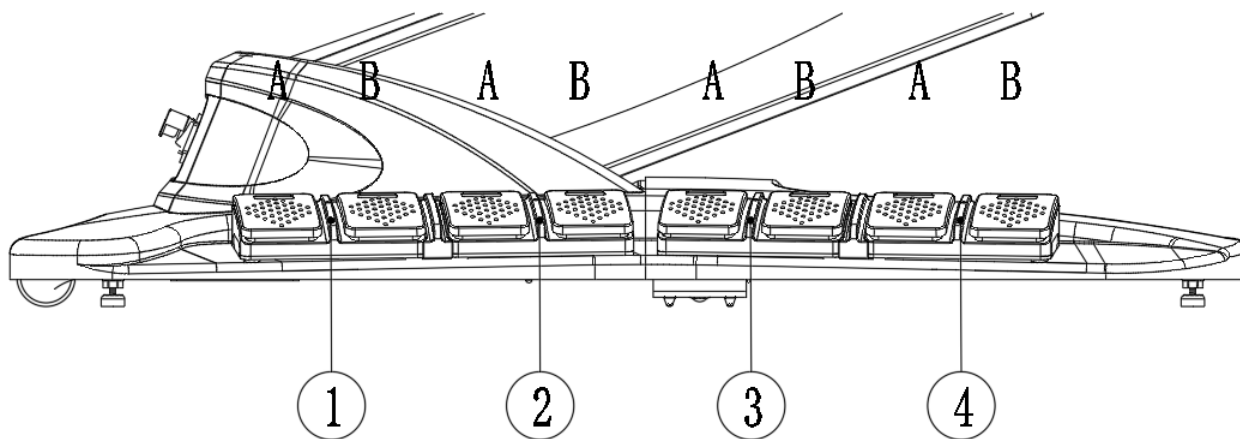


1-Leg cushion adjustment:

Press ④ button A or B  on foot controller or press this button  on leg cushion to adjust tilt angle of leg cushion, as shown in the picture:

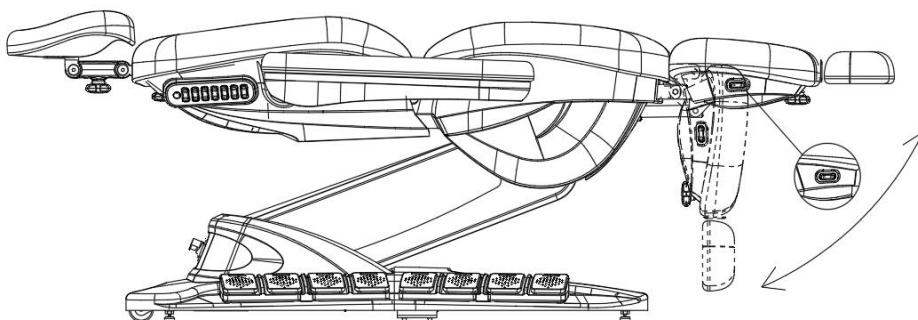


2-Overall adjustment:

Press ③ button A or B  on foot controller to adjust the tilt angle of whole bed, as shown in the picture:

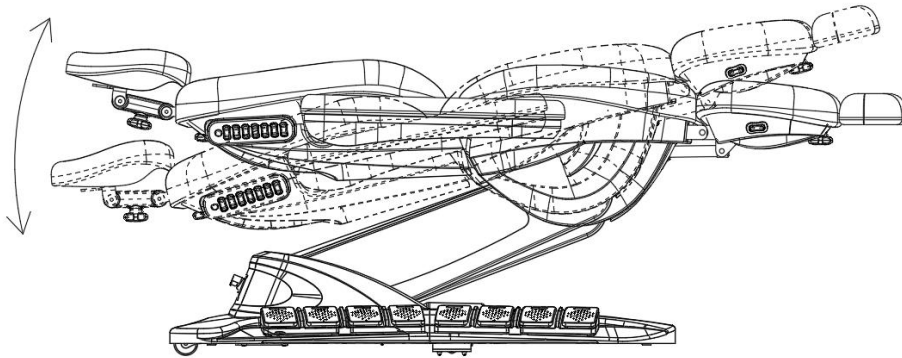


3-Back cushion adjustment:

Press ② button A or B  on foot controller to adjust the tilt angle of back cushion lifting, as shown in the picture:

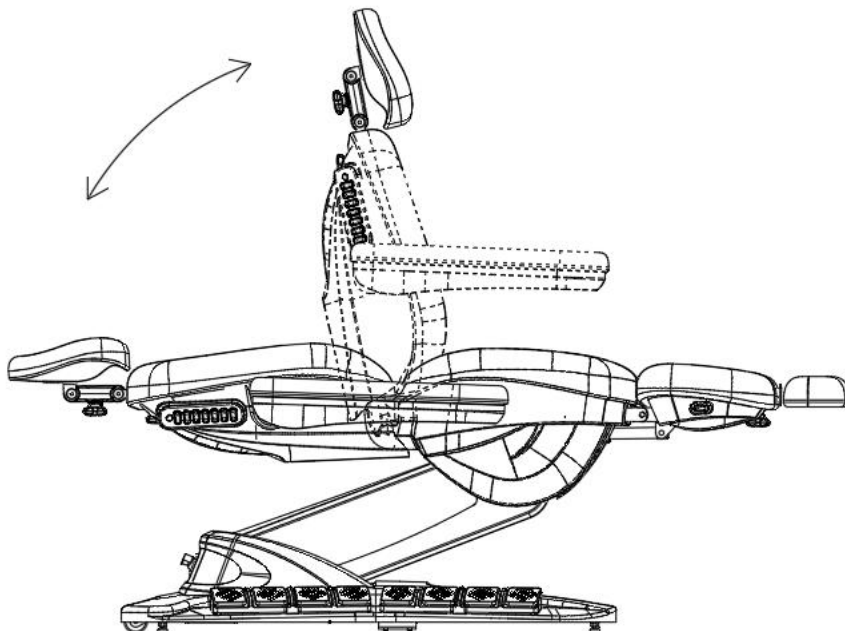


4-Lifting adjustment:

Press ① button A or B  on foot controller to adjust overall lifting of the bed, as shown in the picture:

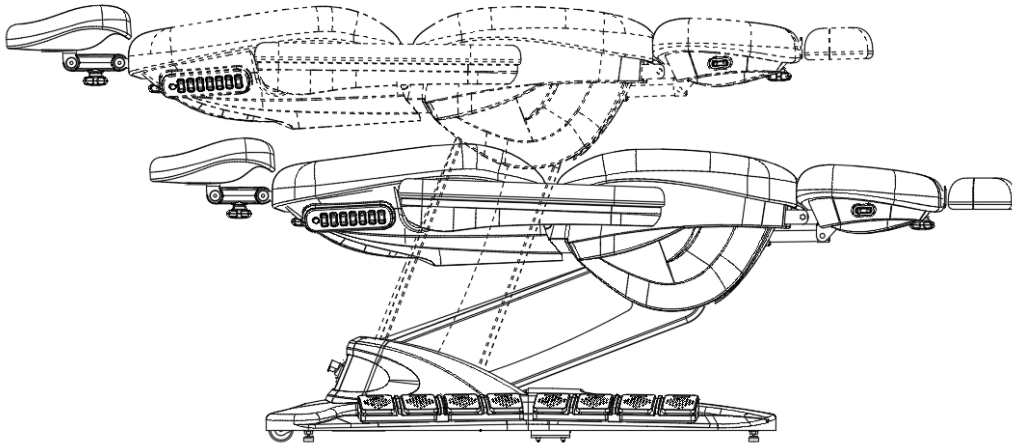


5-Headrest adjustment:

Loose rotary knob A to adjust the height of headrest, unscrew rotary knob B to rotate the headrest freely and turn over angle of it, as shown in the picture:

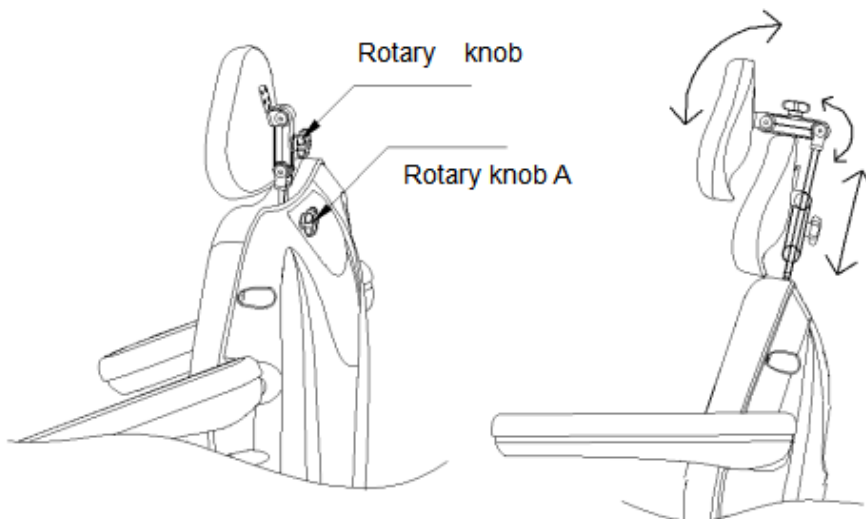


6-Leg cushion adjustment:

Loose rotary knob under the leg cushion to extend it, as shown in the picture:

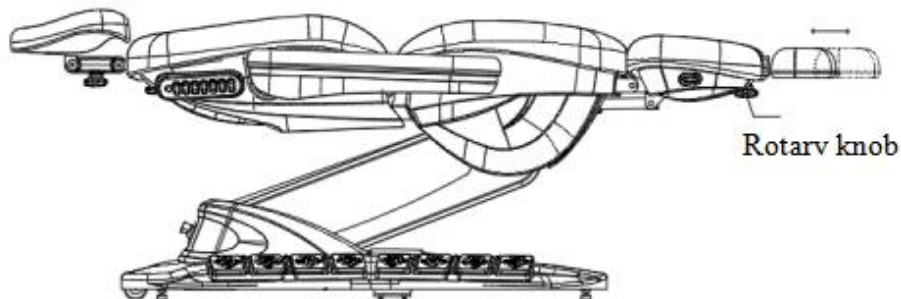


7-Rubber foot adjustment:

Rotate rubber foot to adjust height to keep the bed balance on uneven ground. Tighten and lock the nut after adjusting height, as shown in the picture:

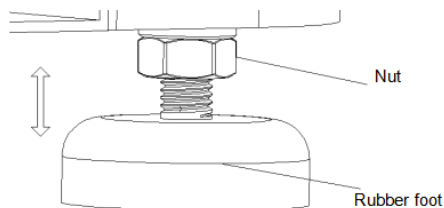

8-Usage of wheels:

Pull up the bed to make the wheels as fulcrum and the bed slope toward the ground, then the whole bed can be moved easily by rotating the wheels, as shown in the picture:

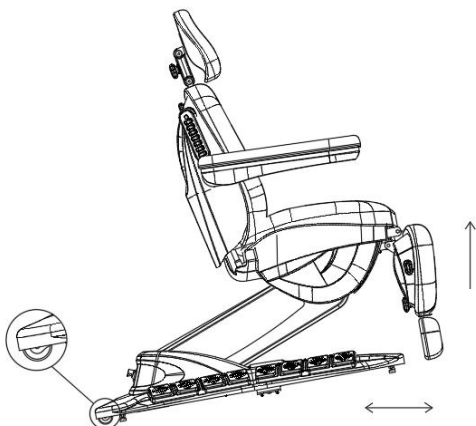

● Maintenance:

1. Daily cleaning

You can wipe out the dust by using soft dry towel.

2. Remove stubborn dirt

It is hard to remove the stubborn dirt by using normal dry towel. In this case, you can use soft towel with diluted neutral detergent to clean it first and then use dry towel to wipe it.

3. It's prohibited to use benzene, diluent, gasoline or other solvent. These solvents may damage the leather or cause corrosion to the surface of this product.

4. Storage and operating environment

Don't put the product under the strong sunlight and moisture environment. Otherwise it may cause damages of the product.

5. It is allowed to use this product under the temperature from -5°C to 50°C and the humidity from 45% to 85%.

● Fault Diagnosis:

If there is something wrong when using the product, please try below operations to detect and eliminate the faults:

Phenomenon	Diagnosis	Operation of elimination
Press the buttons of foot controller but without responses	1.Check whether the controller power switch turn on or not	1.Turn on the controller switch again
	2.Check whether the foot control function has reached to maximum or not	2.Try whether other buttons workable or not
	3.Check whether the power plug is connected or not	3.Unplug the power plug and re-plug it
	4.Check whether each connecting plug of the transformer is connected or not	4.Unplug the connecting plugs and re-plug them
There is noises and shaking during lifting of the bed	Screw loosen	Tighten all the screws
Power switch light does not light up	Fuse has burned out	Replace the fuse

- ◆ It will be not informed if there is any improvement of the product that do not influence basic structure, function and appearance.



Imported by
WEELKO BARCELONA SL

Pol. Ind. La Vall dan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain T.
+34 93 824 90 20
E. info@weelko.com